




- 1**
- Draw air into the syringe up to the red line.
  - Attach the vial to the syringe.
  - Plunge all the air into the vial.

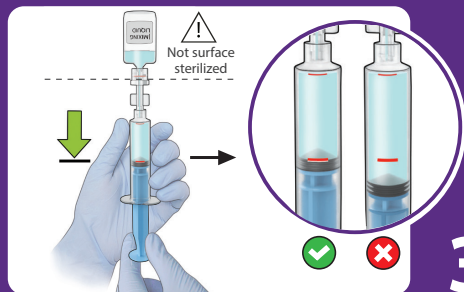


- 2**
- Withdraw liquid up to the red mark on the syringe. There should be saline left in the vial.
  - Disconnect the rotating adapter and vial from the syringe.

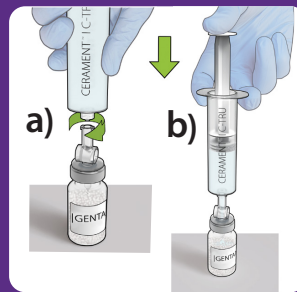


- 3**
- a) Attach the syringe to the spike.
  - b) Inject all solution into the vial.

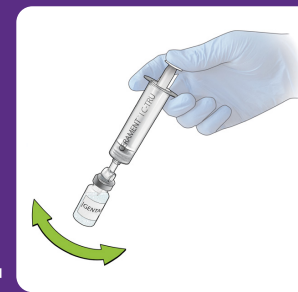


- 4**
- Dissolve all the gentamicin powder.

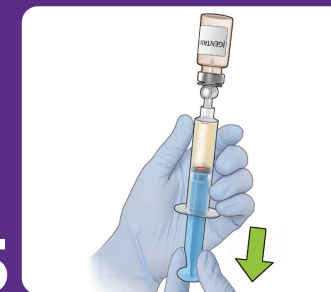


- 5**
- Withdraw all the gentamicin solution back into the syringe. This may require the spike to be slightly backed out.

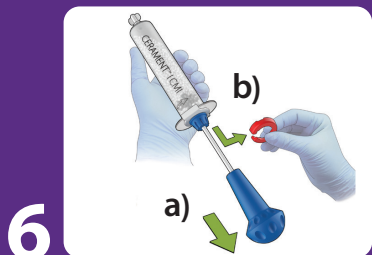


- 6**
- Retract the blue handle and remove the red plunger stop ring.

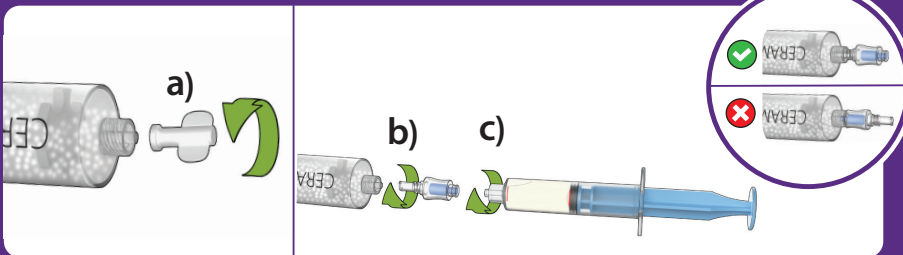


- 7**
- b) Attach valve, clear end to the powder filled syringe, and turn clockwise.
  - c) Attach syringe with gentamicin solution to blue end of valve, press and turn clockwise.

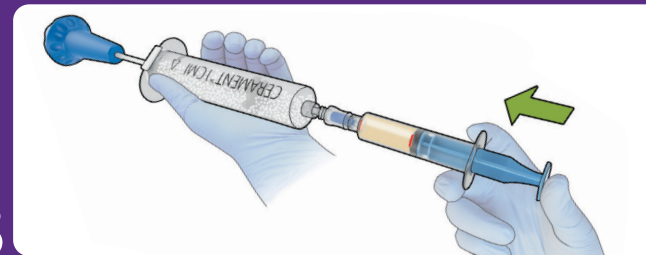


- 8**
- Inject all the gentamicin solution into the syringe with powder.
  - Keep the plunger pushed to the bottom to avoid back flush.
  - Detach the syringe from the valve.

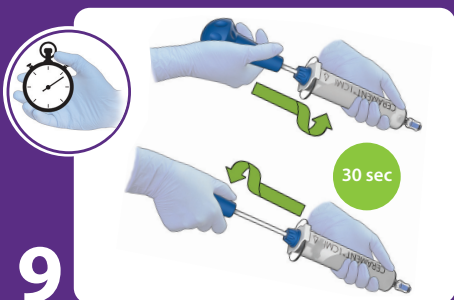


- 9**
- After the liquid has been injected, start the stopwatch immediately.
  - Mix with rotation for 30 seconds, at 1 complete stroke per second.
  - Push the handle all the way down and pull it all the way up, and rotate at its end positions.

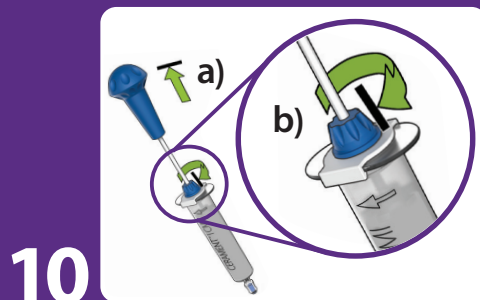

- 10**
- a) Fully retract blue handle.
  - b) Lock plunger by turning blue collar clockwise to click.

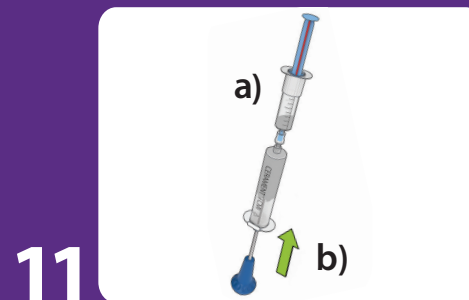

- 11**
- a) Attach dispensing syringe to valve.
  - b) Transfer paste with numbers facing user. When full, paste will begin to ooze from under sleeve. Stop filling when this occurs.

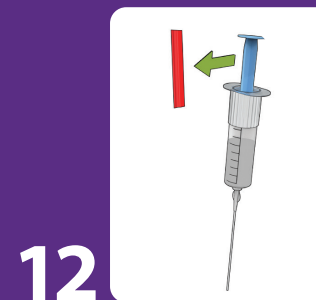

- 12**
- Do not remove the red stopper until ready to inject.

ADDITIONAL RESOURCES:  
Review our mixing video here:

