

CHAPTERS

# 5. Tibial Plateau Fracture

Edited by Dr. R. Schupfner, Klinik für Unfall- und Wiederherstellungschirurgie, Klinikum Bayreuth, Germany

1.  
Benign Bone  
Tumors

2.  
Calcaneus  
Fracture

3.  
White Wound  
Drainage

4.  
Acetabular  
Revision

6.  
High Tibial  
Osteotomy

## CERAMENT®|BONE VOID FILLER in the treatment of Tibial Plateau fractures

### Background:


Proximal tibia fractures are often complex and difficult to treat.

Two main injury mechanisms exist:


- Low energy injuries, often present in elderly, osteoporotic patients after minor trauma
- High energy injuries, usually present in younger patients, e.g. after a fall from height or a motor vehicle accident [1]

### Classification:


AO classification [2]

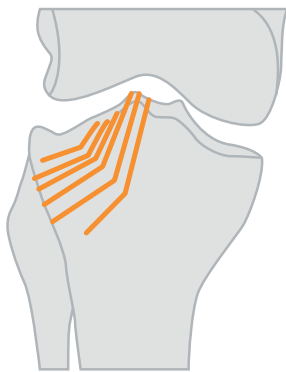


➡ **B2.1**  
Lateral total depression

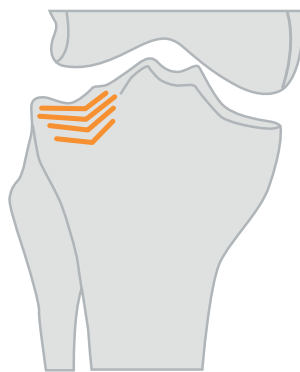


➡ **B2.2**  
Lateral total depression

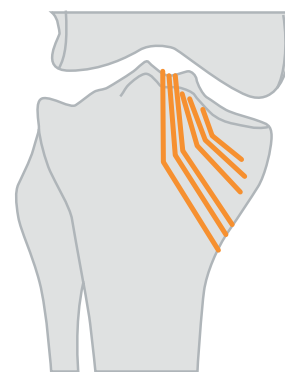


➡ **B2.3**  
Medial depression

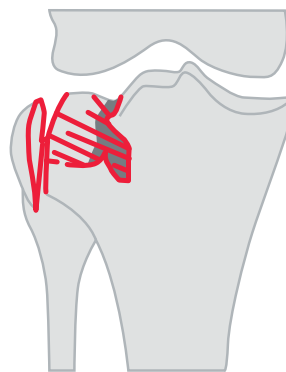

➡ **B3.1**  
Lateral split depression

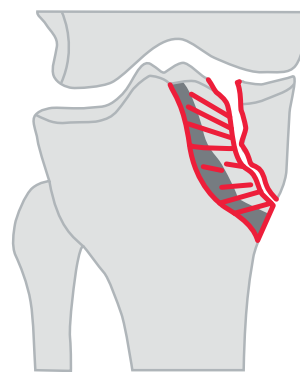

➡ **B3.2**  
Medial split depression

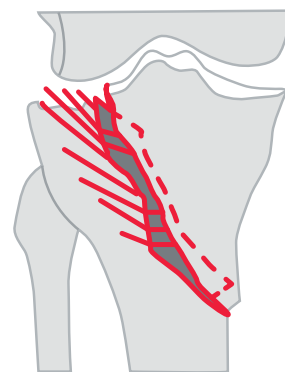

➡ **B3.3**  
Oblique split depression

**Schatzker classification [3]:**



➔ **TYPE I**  
Lateral split



➔ **TYPE II**  
Split with depression



➔ **TYPE III**  
Pure lateral depression



➔ **TYPE IV**  
Pure medial depression



➔ **TYPE V**  
Bicondylar



➔ **TYPE VI**  
Split extends to metadiaphysis

**Diagnostics:**

- ➔ Clinical examination, X-rays AP (Anterior Posterior), and lateral, CT-scan, MRI (ligament injury) [4]

**Therapy:****Conservative treatment:**

- Indication [5]:
- Non-displaced fractures with intact ligaments
  - Stable in varus and valgus stress
  - Low energy trauma with minimal osseous impression
  - Peripheral subminiscal fractures
  - Severe comorbidity of patient
- Therapy:
- Bed rest for 3-4 days with cryotherapy, compression, elevation, pain management and medical antiphlogistic therapy (RICE-therapy: rest, ice, compression, elevation)
  - If splinting is necessary, immobilisation in a hinged fracture brace should be used [6]
  - Start active range of motion exercises as soon as possible
  - No weight bearing or only partial weight bearing up to 10kg for 6–8 weeks
  - Progressive weight bearing should begin at 6–8 weeks, according to radiographic controls

**Operative treatment [6]:**

- All displaced tibial plateau fractures
- Open fractures
- Concomitant compartment syndrome
- Fractures with nerve or vessel injury

**Surgical**

- techniques:
- Closed reduction and minimal invasive internal fixation with cannulated screws [6–8]
  - Closed reduction and external fixation (Ilizarov [9], Taylor Spatial Frame [10], Hybrid external fixation [11])
  - Open reduction and internal fixation (ORIF), usually with a Locking Compression Plate (LCP) [8, 12] or Less Invasive Stabilisation System (LISS) [8, 13]

The reduction of the depressed fragment of the tibia plateau can be achieved by lifting up the fragment with bone punches through a cortical window [7]. This manoeuvre leads to a bone void.

**Treatment options of bone voids**

There is still a lack of evidence to determine the best method for treating the bone defects in tibial plateau fractures [15]. Different treatment options exist, some are listed below:

- ➔ Autologous bone graft [7, 16]
- ➔ Allograft [17]
- ➔ Calcium phosphate bone cement [18, 19, 20]
- ➔ Hydroxyapatite calcium carbonate synthetic bone graft [21]
- ➔ Hydroxyapatite cement [22]
- ➔ Bioactive glass granules [23]
- ➔ Calcium sulphate / Calcium phosphate bone substitute: CERAMENT®|BONE VOID FILLER [24]

## Literature

1. Mills WJ, Nork SE. Open reduction and internal fixation of high energy tibial plateau fractures. *Orthop Clin North Am* 2002; 33: 177-198
2. Müller ME, Nazarian S, Koch P, et al. The Comprehensive Classification of Fractures of Long Bones, 1st ed. 1990, Springer-Verlag, Berlin, Heidelberg, New York
3. Schatzker J, McBroom R, Bruch D. The Tibia plateau fracture: The Toronto experience 1968 – 1975. *Clin Orthop Relat Res* 1979; 138: 94-104
4. Yacoubian SV, Nevins RT, Sallis JG, Potter HG, Lorich DG. Impact of MRI on treatment plan and fracture classification of tibial plateau fractures. *J Orthop Trauma* 2002; 16: 63 –637
5. Stannard JP, Schmidt AH, Kregor PJ. Surgical treatment of orthopaedic trauma. 2007. Thieme. New York
6. Colton C, Krikler S, Schatzker J, Trafton P. AO-Foundation. AO Surgery Reference. [www2.aofoundation.org/wps/portal/surgery](http://www2.aofoundation.org/wps/portal/surgery)
7. Duwelius PJ, Rangitsch MR, Colville MR, Woll TS. Treatment of tibial plateau fractures by limited internal fixation. *Clin Orthop Relat Res*. 1997; 339: 47-57
8. Rüedi TP, Buckley RE, Moran CG [Edts.] AO Principles of fracture management. 2007. Thieme. New York
9. Catagni MA, Ottaviani G, Maggioni M. Treatment strategies for complex fractures of the tibial plateau with external circular fixation and limited internal fixation. *J Trauma*. 2007; 63: 1043-1053
10. Ahearn N, Oppy A, Halliday R, Rowett-Harris J, Morris SA, Chesser TJ, Livingstone JA. The outcome following fixation of bicondylar tibial plateau fractures. *Bone Joint J*. 2014; 96-B: 956-962
11. Babis GC, Evangelopoulos DS, Kontovazenitis P, Nikolopoulos K, Soucacos PN. High energy tibial plateau fractures treated with hybrid external fixation. *J Orthop Surg Res*. 2011; 6:35-42
12. Gosling T, Schandelmaier P, Muller M, Hankemeier S, Wagner M, Krettek C. Single lateral locked screw plating of bicondylar tibial plateau fractures. *Clin Orthop Relat Res*. 2005; 439: 207-214
13. Cole PA, Zlowodzki M, Kregor PJ. Treatment of proximal tibia fractures using the less invasive stabilization system: surgical experience and early clinical results in 77 fractures. *J Orthop Trauma*. 2004; 18: 528-535
14. Buchko GM, Johnson DH. Arthroscopy assisted operative management of tibial plateau fractures. *Clin Orthop Relat Res*. 1996; 332: 29-36
15. Giannoudis PV, Dinopoulos H, Tsiridis E. Bone substitutes: an update. *Injury*. 2005; 36 Suppl 3: S20-27
16. Segal D, Franchi AV, Campanile J. Iliac autograft for reconstruction of severely depressed fracture of a lateral tibial plateau. Brief note. *J Bone Joint Surg Am*. 1985; 67: 1270-1272
17. Segur JM, Torner P, García S, Combalá A, Suso S, Ramón R. Use of bone allograft in tibial plateau fractures. *Arch Orthop Trauma Surg*. 1998; 117: 357-359
18. Horstmann WG, Verheyen CC, Leemans R. An injectable calcium phosphate cement as a bone-graft substitute in the treatment of displaced lateral tibial plateau fractures. *Injury*. 2003; 34: 141-144
19. Lobenhoffer P, Gerich T, Witte F, Tscherne H. Use of an injectable calcium phosphate bone cement in the treatment of tibial plateau fractures: A prospective study of twenty-six cases with twenty month mean follow up. *Orthop Trauma* 2002; 16: 143 – 146
20. Russell TA, Leighton RK. Comparison of autogenous bone graft and endothermic calcium phosphate cement for defect augmentation in tibial plateau fractures. A multicenter, prospective, randomized study. *J Bone Joint Surg Am*. 2008; 90: 2057-2061
21. Ong JC, Kennedy MT, Mitra A, Harty JA. Fixation of tibial plateau fractures with synthetic bone graft versus natural bone graft: a comparison study. *Ir J Med Sci*. 2012; 181: 247-252
22. Dickson KF, Friedman J, Buchholz JG, Flandry FD. The use of BoneSource hydroxyapatite cement for traumatic metaphyseal bone void filling. *J Trauma*. 2002; 53: 1103-1108.
23. Heikkilä JT, Kukkonen J, Aho AJ, Moisander S, Kyyrönen T, Mattila K. Bioactive glass granules: a suitable bone substitute material in the operative treatment of depressed lateral tibial plateau fractures: a prospective, randomized 1 year follow-up study. *J Mater Sci Mater Med*. 2011; 22: 1073-1080
24. Iundusi R, Gasbarra E, D'Arienzo M, Piccioli A, Tarantino U. Augmentation of tibial plateau fractures with an injectable bone substitute: CERAMENT®. Three year follow-up from a prospective study. *BMC Musculoskelet Disord*. 2015; 16: 115.

## CERAMENT®|BONE VOID FILLER in the treatment of Tibial Plateau fractures

Open reduction and internal fixation with cannulated screws, a Locking Compression Plate (LCP) or the Less Invasive Stabilisation System (LISS)

### Surgical positioning and preoperative procedures:

- ➔ Mark the site of surgery while informed consent of patient is obtained
- ➔ The use of a radiolucent table is recommended
- ➔ Prepare mobile C-arm
- ➔ Antibiotic prophylaxis 30 min before incision [1]
- ➔ Place the patient in a supine position with a pillow under the ipsilateral knee
- ➔ Place a pneumatic thigh tourniquet and activate it
- ➔ Skin preparation and draping as usual
- ➔ Team time-out

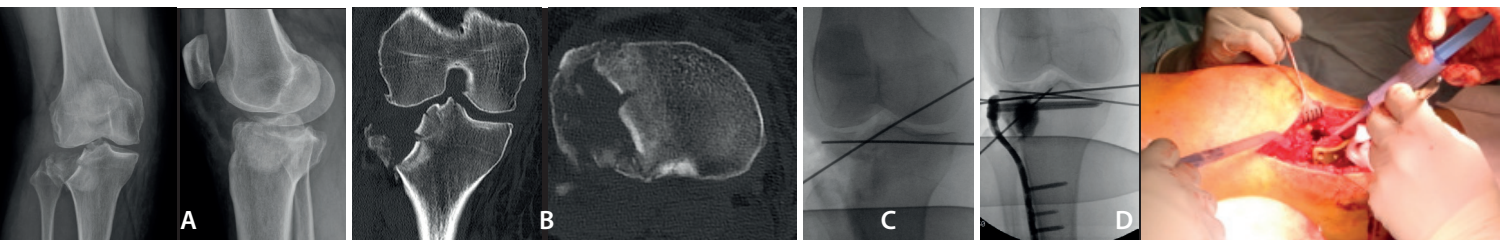


Figure 1.

Pre-operative X-rays in A/P and lateral view (A) and CT scans in A/P and sagittal view (B) demonstrating an AO grade 41-B3.1 fracture; temporal fixation of lifted fragment by K-wires (C); final fixation with LCP-plate and filling of residual bone void with CERAMENT®|BONE VOID FILLER via a lateral cortical window (D)

Images reproduced by kind permission of Dr. R. Schupfner, Klinik für Unfall- und Wiederherstellungschirurgie, Klinikum Bayreuth, Germany

### Surgery:


- ➔ Usually an anterolateral, parapatellar approach is used.
- ➔ Via this approach a lateral arthrotomy enables a direct view of the lateral joint surface.
- ➔ A lateral cortical window is prepared. Four holes in the shape of a 1 cm<sup>2</sup> are drilled using a 2 mm drill bit. The drill holes are connected with a small osteotome and the cortical bone window opened.
- ➔ The depressed fragment is now lifted up using a bone punch through the cortical window.
- ➔ Temporal fixation of the fragment realized by K-wires. (Fig. 1C).
- ➔ For internal fixation cannulated screws, a LCP or the LISS can be used.
- ➔ In case of screws 3.5 mm [2] up to 6.5 mm [3] cannulated screws (with washers) are placed over K-wires.
- ➔ In case of a plate usually an anatomically shaped LCP is used according to the suggested surgical technique [4,5].
- ➔ If the LISS is used, follow the established surgical instructions [5,6].
- ➔ Fracture reduction and implant placement should be controlled by fluoroscopy.
- ➔ As a next step, the bone void, which resulted from the reduction of the fragment, is filled.
- ➔ Mix CERAMENT®|BONE VOID FILLER as per Instructions for use.
- ➔ Wait for three minutes until the material will be more viscous.
- ➔ Inject CERAMENT®|BONE VOID FILLER in the void in backfill technique under fluoroscopic control. (Fig. 1D).
- ➔ The cortical bone window can be closed if appropriate.
- ➔ Wait for 10 minutes until CERAMENT®|BONE VOID FILLER has hardened.
- ➔ The surgical tourniquet can be released and accurate hemostasis should be achieved.
- ➔ If required a drain with contact to the cortical window can be used.
- ➔ Perform a layered wound closure.



## CERAMENT®|BONE VOID FILLER in the treatment of Tibial Plateau fractures:

### Follow Up:

- ➔ Clinical and radiographic controls

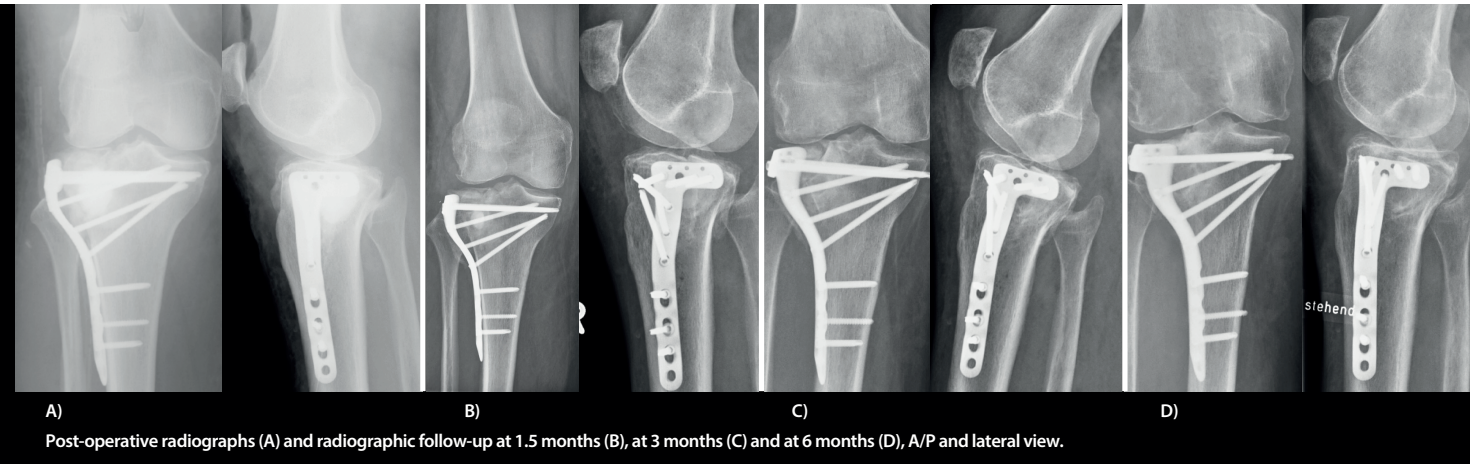

Images reproduced by kind permission of Dr. R. Schupfner, Klinik für Unfall- und Wiederherstellungschirurgie, Klinikum Bayreuth, Germany



- ➔ Ensure good contact with cancellous bone
- ➔ Wait three minutes after mixing before you start to inject CERAMENT®|BONE VOID FILLER ('Spaghetti-test')
  - Control bleeding during surgery
  - Extensive bleeding might result in intermixing of blood with the CERAMENT® paste
  - Consider using a tourniquet
- ➔ Follow normal surgical practice and if applicable use a drain with contact to the hardened CERAMENT®
  - The drain may draw white coloured fluid some hours after surgery, which does not endanger the success of surgery
- ➔ Close soft tissue and skin in layers:
  - Complete all deep sutures first before tightening them in a second step

## Literature

1. Bratzler DW, Houck PM. Antimicrobial prophylaxis for surgery: an advisory statement from the National Surgical Infection Prevention Project. Clin Infect Dis. 2004; 38: 1706-1715
2. Stannard JP, Schmidt AH, Kregor PJ. Surgical treatment of orthopaedic trauma. 2007. Thieme. New York
3. Rüedi TP, Buckley RE, Moran CG [Edts.] AO Principles of fracture management. 2007. Thieme. New York
4. Gosling T, Schandelmaier P, Muller M, Hankemeier S, Wagner M, Krettek C. Single lateral locked screw plating of bicondylar tibial plateau fractures. Clin Orthop Relat Res. 2005; 439: 207-214. [www.synthes.com/sites/na/products/trauma/Pages/home.aspx](http://www.synthes.com/sites/na/products/trauma/Pages/home.aspx)
5. Cole PA, Zlowodzki M, Kregor PJ. Treatment of proximal tibia fractures using the less invasive stabilization system: surgical experience and early clinical results in 77 fractures. J Orthop Trauma. 2004; 18: 528-535