

CERAMENT® Bead Tray

PRODUCT SHEET

MIX. MOLD. DELIVER.

The CERAMENT® Bead Tray provides an adjunctive local delivery option for surgeons desiring targeted delivery to osseous defects and for dead space management.

- Made of flexible, medical-grade thermoplastic, making it easy to fill, set, and release beads
- Three distinct bead sizes provide ultimate control over targeted delivery
- Supplied sterile and intended for single-use
- 10ml of CERAMENT will fill all bead cavities

BEAD SIZE	VOLUME
3 mm	72 beads
4.7 mm	60 beads
6 mm	30 beads

CERAMENT® Bead Tray is a delivery device for:

CERAMENT® BONE VOID FILLER

CERAMENT® G with Gentamicin



CERAMENT® G:

- Bone remodeling to promote^{1,2} and protect bone healing^{1,2}
- Local antibiotic elution that is safe, consistent and clinically significant³

1. Hofmann et al. 'Autologous Iliac Bone Graft Compared with Biphasic Hydroxyapatite and Calcium Sulfate Cement for the Treatment of Bone Defects in Tibial Plateau Fractures: A Prospective, Randomized, Open-Label, Multicenter Study'. *J Bone Joint Surg Am.* 2020;102(3):179-193.

2. Ferguson et al. 'Radiographic and Histological Analysis of a Synthetic Bone Graft Substitute Eluting Gentamicin in the Treatment of Chronic Osteomyelitis'. *J Bone Joint Infect.* 2019;4:76-84.

3. Stravinkas et al. 'Pharmacokinetics of gentamicin eluted from a regenerating bone graft substitute - in vitro and clinical release studies'. *Bone Joint Res.* 2016; 5:427-435

CERAMENT® Bead Tray

PRODUCT SHEET

MIX. MOLD. DELIVER.

Directions for use:

- Mix CERAMENT G or CERAMENT BONE VOID FILLER according to their instructions for use
- Extrude the bone graft substitute paste into the bead mold
- Using the spatula, spread the material evenly, completely filling each bead cavity
- Remove air bubbles by gently tapping the bead mold on a flat surface
- Let the beads set for 15 minutes before releasing
- Flex the mold to release the beads
- Gently pack the beads into the treatment site
- Avoid overfilling or compressing the treatment site
- Close the wound(s) meticulously to avoid leakage into soft tissue



PRODUCT CODES

DESCRIPTION	ARTICLE NUMBER
CERAMENT® Bead Tray	A0513
CERAMENT® G with Gentamicin 5ml	A0535-06
CERAMENT® G with Gentamicin 10ml	A0535-05
CERAMENT® BONE VOID FILLER 5ml	A0210-09
CERAMENT® BONE VOID FILLER 10ml	A0210-08
CERAMENT® BONE VOID FILLER 18ml	A0210-11

Availability of CERAMENT® is dependent on regulatory status in individual markets, contact your local representative. For complete product information, including indications, contraindications, warnings, precautions and potential adverse events, see package insert.



BONESUPPORT, INC.,
117 Fourth Ave, Suite 202
Needham, MA 02494

T: 1.877.719.6718
E: us.sales@bonesupport.com
W: www.bonesupport.com

